

# ANIOSYME 5 SYNERGY

5 ENZYMES FOR 1 UNIQUE PRODUCT

- 5-enzyme detergent with broad-spectrum performance
- Multi-application: soaking, ultrasonic bath, washer-disinfector
- Effective after 1 minute in washer-disinfector
- Effective in all qualities of water (softened, mains, etc.)
- Product not classed as dangerous for the environment
- VOC-free formulation, neutral pH at use dilution

## INDICATIONS

Cleaning of surgical and medical equipment and medical devices in soaking, ultrasonic bath, automatic washing machine and washing tunnel.

ANIOSYME SYNERGY 5, with its 5 enzymes (amylase, cellulase, lipase, mannanase and protease), provides broad-spectrum performance on the ingredients of all kinds of soil.

## CHARACTERISTICS

- Viscosity (pure product): 10 mPa.s
- Density: about 1.115
- pH (pure product): about 8
- pH (diluted product): about 7.25 (neutral)
- Tested in accordance with the available cleaning indicators:
  - Cleaning/disinfection plan on different types of soil in soaking (fat and blood)
  - On-board tests in washer-disinfectors
  - ATPmetry
  - So-called "German" soil described by EN 15883 standard in ultrasonic bath

## COMPOSITION

Non-ionic and anionic tensioactive agents, sequestering agent, stabilising agent, enzyme complex, excipients.

## INSTRUCTIONS FOR USE

Use at dilutions of between 0.05 and 0.5%.  
Adjust the concentration, contact time and temperature of the dilution water in accordance with the level of soil and cleaning method used.  
Brush if necessary. Rinse thoroughly.  
Maximum temperature: 60°C.

## PRECAUTIONS FOR USE

Dangerous - follow the precautions for use (established in accordance with Directive 99/45/EC and its adaptations).  
Store between 5°C and 35°C.  
Class I medical device (amended Directive 93/42/EEC)



## PACKAGING

Use for soaking or ultrasonic bath

12 bottles 1L measure  
4 x 5L drums + 25 ml pump

Ref. 2235.095  
Ref. 2235.036

Use in machine

2 x 5L drums

Ref. 2235.038



Pavé du Moulin  
59260 Lille-Hellemmes - France

[www.anios.com](http://www.anios.com)



EXCLUSIVE DISTRIBUTOR (Sole Agent)  
**PT. PRIMA ALKESINDO NUSANTARA**  
Komplek Perkantoran Gedung Bakti Indah Blok C No. 12-15,  
Jl. Bukit Gading Raya, Kelapa Gading Permai, Jakarta 14240  
Telp. : (021) 4513814 - 75 Fax. : (021) 4513878  
E-mail : [primad@alkesindo.id](mailto:primad@alkesindo.id)  
Website : [www.primaalkesindo.com](http://www.primaalkesindo.com)